



The XC9900PK purge adapter kit should be used by installers to provide a nitrogen purge thru the medical gas outlets while brazing the outlets to the medical gas pipeline in accordance with NFPA 99.

Following the installation instructions provided with the Tri-Tech Medical X series medical gas outlets, the XC9900PK kit includes; an adapter, mounting screws, a one foot piece of tubing and a tubing x ¼ M npt adapter which should be fastened to a N2 (nitrogen) hose to provide a constant N2 purge thru the outlet while it is being brazed to the medical gas pipeline.



XC9900PK Outlet Purge Kit



Kit attached to rough-in of medical gas outlet

How to Order

| Part # | Description |
|---------------|---------------------------------------------------------------------------------------------|
| XC9900PK | Outlet purge kit for Tri-Tech Medical X Series Outlets – all styles, types and gas services |